D Reinigung

Cleaning GB

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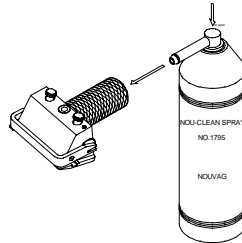
- **Do not clean the Dermotome with compressed air.**
- **Autoclaving the instrument without packing may seriously damage it!**

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- 1) Remove bottom plate and blade after each procedure.
- 2) Dispose blunt blade.
- 3) The Dermotome, bottom plate and blade should be cleaned after each use to prevent the build up of deposits and debris that can destroy them, if not removed. Wipe them with a clean cloth dampened with disinfectant solution. The instrument can be scrubbed under cold running water, as well.
- 4) Dry bottom plate and blade with clean cloth or paper.
- 5) Apply some high performance oil drops to bottom plate and blade, reassemble.
- 6) Apply some oil drops to all the movable parts, as well.
- 7) Spray the Dermotome interior with **NOU-CLEAN spray** → **see spray can instructions.**
- 8) Pack the Dermotome into a sterilization foil.
- 9) After packing Dermotome, autoclave **at max. 134°C.**

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After autoclaving, allow instrument to dry for one hour at room temperature.

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Problem:	Cause:	Solution:
Dermatome does not cut	Device is not assembled properly	Check blade fitting
Motor runs, but blade does not move	Instrument is not connected properly to the motor	Press Dermotome firm on motor until instrument snaps in. Check fitting

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Art.-Nr.	Description	Units
1995	Blade 75 mm	10 pieces
1996	Blade 50 mm	10 pieces
1997	Blade 25 mm	10 pieces
1998	Blade 12 mm	10 pieces
1882	Oil bottle	1 piece
1984	Nou-Clean Spray	1 piece
1958	Nozzle for Nou-Clean spray	1 piece
19935	Reduction support base 75mm to 50mm	1 piece
19936	Reduction support base 75mm to 25mm	1 piece

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Disposal:

Disposal of device, components and accessories must strictly conform to local laws and regulations as set out by the relevant authorities.

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NOUVAG AG, St. Gallerstrasse 23-25, CH-9403 Goldach
NOUVAG GmbH, Schulthaiss-Strasse 15, D-78462 Konstanz
Robbins Instruments Inc, 2 North Passaic Ave, Chatham, New Jersey 07928